

YOUNG® PerioDT™

WITH DESICCATION TECHNOLOGY

Application Process

STEP 01 DRY

Dry the surface to be treated with air or gauze.

STEP 02 APPLY

Apply PerioDT subgingivally until product flows from the pocket, leaving a thin layer supragingivally. Rinse after 10-30 seconds.

STEP 03 EVACUATE & RINSE

Thoroughly remove PerioDT with a high-speed evacuator followed by a water rinse.

STEP 04 INSTRUMENTATION

Perform scaling and root planing treatment.

STEP 05 REAPPLY

As a final step, repeat PerioDT application steps 1-3 to destroy any remaining biofilm.

NOTE: When utilizing PerioDT for periodontal treatment and/or heavy debridement, it is recommended to thoroughly evacuate and rinse all PerioDT before beginning your scaling procedure. Then, if needed, reapply to periodontally diseased pockets and follow the instructions for use. **Additional adjunct treatments are not necessary or recommended following PerioDT application. Additional medicaments placed in periodontal pockets may increase the risk of post treatment irritation or inflammation. Patients may resume normal home care routines and there are no dietary restrictions immediately post treatment.**

Reference Instructions for Use for additional application tips and safety information at bit.ly/periodt-ifu

Indication and Workflow for Periodontal Treatment

Advanced Periodontitis



Dry the Surface



Apply PerioDT
Supragingivally and
Subgingivally



Evacuate and Rinse



Scaling and Root Planing



Dry the Surface



Reapply PerioDT



Evacuate and Rinse